EN FR DE IT

# FertiCult™ Flushing medium

## FertiCult™ Flushing medium

Cell culture medium for washing of human ova, spermatozoa and embryos, for swim-up of spermatozoa, sperm injection in ICSI, intra uterine insemination and embryo transfer

STERILE A

Document reference: FP09 I08 R01 C.7

Update: 04.12.2020

### USED ABBREVIATIONS

ICSI Intracytoplasmatic Sperm Injection  
IUI Intra Uterine Insemination

### INDICATIONS FOR USE

FertiCult™ Flushing medium is intended for *in vitro* procedures involving human gametes (sperm and oocytes), including washing of gametes, sperm swim-up procedures, intra-uterine insemination (IUI) of the spermatozoa and sperm injection during intracytoplasmic sperm injection (ICSI). FertiCult™ Flushing medium can also be used for human embryo washing and holding, and for embryo transfer (ET) in the uterine cavity.  
*For professional use only.*

### GENERAL INFORMATION

The medium is complete and needs no further additives. The medium contains HEPES and CO<sub>2</sub> incubation is required. FertiCult™ Flushing medium does not contain heparin, thus for oocyte pick-up, use FertiCult™ Aspiration medium.

### COMPOSITION

FertiCult™ Flushing medium is a ready-to-use HEPES-buffered medium which also contains bicarbonate, physiologic salts, glucose, lactate and human serum albumin (4g/l). The product is also available with phenol red and/or gentamicin (10mg/l).

### MATERIALS INCLUDED

Product code: FLUSH020  
» 5x 20ml FertiCult™ Flushing medium  
Product code: FLUSH2.8  
» 1x 22,8ml FertiCult™ Flushing medium  
Product code: FLUSH050  
» 5x 50ml FertiCult™ Flushing medium  
Product code: FLUSH100  
» 3x 100ml FertiCult™ Flushing medium  
Product code: FLUSH\_002\_PHR  
» 1x 2ml FertiCult™ Flushing medium with phenol red  
Product code: FLUSH020PHR  
» 5x 20ml FertiCult™ Flushing medium with phenol red  
Product code: FLUSH100PHR  
» 3x 100ml FertiCult™ Flushing medium with phenol red  
Product code: FLUSH500PHR  
» 1x 500ml FertiCult™ Flushing medium with phenol red  
Product code: FLUSH020PRG  
» 5x 20ml FertiCult™ Flushing medium with Phenol Red and Gentamicin  
Product code: FLUSH050PRG  
» 5x 50ml FertiCult™ Flushing medium with Phenol Red and Gentamicin  
Product code: FLUSH100PR\_G  
» 3x 100ml FertiCult™ Flushing medium with Phenol Red and Gentamicin  
Product code: FLUSH500PRH\_G  
» 1x 500ml FertiCult™ Flushing medium with Phenol Red and Gentamicin  
Product code: WASH005  
» 1x 5ml Washing/Insemination medium  
Product code: WASH020  
» 1x 20ml Washing/Insemination medium  
Product code: WASH100G  
» 1x100ml Washing/Insemination medium with Gentamicin

### MATERIALS NEEDED BUT NOT PROVIDED

» Incubator at 37°C (No CO<sub>2</sub>)  
» Petri dishes  
» Microscope  
» Test tubes  
» LAF bench (ISO Class 5)  
» Syringe (e.g. 1ml Plastipack)  
» Catheter

### PRODUCT SPECIFICATIONS

» Chemical composition:  
pH release criteria: 7.30-7.60  
pH stability criteria: 7.30-7.90  
» Osmolality: 270-290 mOsm/kg  
» Sterility test by the current Ph. Eur. 2.6.1 / USP <71>: No growth  
» Endotoxin test by Limulus Amebocyte Lysate (LAL) methodology (USP <85>); < 0.25 EU/ml  
» 1-Cell Mouse Embryo Assay (blastocyst after 96h); ≥ 80% after 60min of exposure (zygote stage)  
» Use of Ph Eur or USP grade products if applicable  
» Certificate of analysis and MSDS are available upon request

### WARNINGS BEFORE USE

» Do not use the product if:  
- it becomes discoloured (if medium contains phenol red), cloudy or shows any evidence of microbial contamination  
- seal of the container is opened or defect when the product is delivered  
- expiry date has been exceeded  
» Do not freeze before use  
» Do not re-sterilize after opening  
» Products that include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.

### STORAGE INSTRUCTIONS

» Store FertiCult™ Flushing medium without gentamicin between 2-25°C.  
» Store FertiCult™ Flushing medium with gentamicin between 2-8°C.  
» Once opened store all products between 2-8°C.  
» Keep away from sunlight.  
» The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8°C.  
» Stable after transport (max. 5 days) at elevated temperature (≤ 37°C).

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» 3x 100ml FertiCult™ Flushing medium

Code produit: FLUSH002\_PHR

» 1x 2ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH020PHR

» 5x 20ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH100PHR

» 3x 100ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH500PHR

» 1x 500ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH002\_PHR

» 1x 2ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH020PRO

» 5x 20ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH100PRO

» 3x 100ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH500PRO

» 1x 500ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH002\_PHR

» 1x 2ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH020PRO

» 5x 20ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH100PRO

» 3x 100ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH500PRO

» 1x 500ml FertiCult™ Flushing medium with phenol red

Product code: FLUSH005

» 1x 5ml Washing/Insemination medium

Code produit: WASH020

» 1x 20ml Washing/Insemination medium

Code produit: WASH100G

» 1x100ml Washing/Insemination medium with Gentamicin

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