## FertiCult™ IVF medium

**INDICATIONS FOR USE**

FertiCult™ IVF medium is intended for washing and holding of human oocytes, performing fertilization by IVF or ICSI (until 2PN).

*For professional use only.*

### GENERAL INFORMATION

The medium is complete and needs no further additives. As with all IVF media, FertiCult™ IVF medium **has to be pre-incubated in a CO2 incubator with 5-6% CO2 for at least 5 hours, but ideally, dishes should be prepared the day before use and incubated overnight** (with lid opened) to obtain an optimal pH for oocyte washing/holding.

### COMPOSITION

FertiCult™ IVF medium is a ready-to-use bicarbonate-buffered balanced salt solution, supplemented with glucose (5.55mM), lactate, pyruvate, and human serum albumin (4.0g/l; medicinal substance derived from human blood plasma). The product is also available with phenol red and/ or gentamicin (10mg/l; medicinal substance).

### PRODUCT SPECIFICATIONS

- Chemical composition
- pH: 7.20 – 7.50 (37 °C – 6% CO2)
- Osmolality: 270 - 290 mOsm/kg
- Sterility test by the current Ph. Eur. 2.6.1. / USP <71>: No growth
- Endotoxin test by Limulus Ameobocyte Lystate (LAL) methodology (USP <85>): < 0.25EU/ml
- Mouse Embryo Assay (% blastocysts after 96h after IVF in test medium): ≥ 80%
- Use of Ph Eur or USP grade raw materials if applicable Certificate of analysis and MSDS are available upon request or can be downloaded from our website ([www.fertipro.com](http://www.fertipro.com)).

### STERILE

Document ID: FP09 I07 R01 F.2

Update: 07/12/2023

<b>MATERIAL INCLUDED/</b>	<b>MATÉRIEL INCLUS/</b>
<b>IM LIEFERUMFANG ENTHALTENE MATERIALIEN/</b>	<b>MATERIALI INCLUIDO/</b>
<b>MATERIALI INCLUSI/</b>	<b>MATERIALI INCLUIDO/</b>
<b>ΥΑΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/</b>	<b>MATERIAAL AANWEZIG</b>
<b>Kit components</b>	<b>Contenido del kit</b>
<b>Catalogue number</b>	<b>Referencia catalogue</b>
<b>Reference number</b>	<b>Referenznummer</b>
<b>Beginning number</b>	<b>Número de catálogo</b>
<b>Number of catalog</b>	<b>Número de catálogo</b>
<b>Αριθμός καταλόγου</b>	<b>Aριθμός καταλόγου</b>
<b>Αριθμός τετρακάβου</b>	<b>Catálogo número</b>

FertiCult™ IVF medium		
FEQJ020	5 x 20ml	FertiCult™ IVF medium
FEQJ050	5 x 50ml	FertiCult™ IVF medium
FEQJ100	3 x 100ml	FertiCult™ IVF medium
FertiCult™ IVF medium with phenol red		
FEQJ020PHR	5 x 20ml	FertiCult™ IVF medium with phenol red
FEQJ050PHR	5 x 50ml	FertiCult™ IVF medium with phenol red
FEQJ100PHR	3 x 100ml	FertiCult™ IVF medium with phenol red
FertiCult™ IVF medium with phenol red and gentamicin		
FEQJ020PHR_G	5 x 20ml	FertiCult™ IVF medium with phenol red and gentamicin
FEQJ050PHR_G	5 x 50ml	FertiCult™ IVF medium with phenol red and gentamicin
FEQJ100PHR_G	3 x 100ml	FertiCult™ IVF medium with phenol red and gentamicin

<sup>1</sup> Conaghan J (2014). pH control in the embryo culture environment. In: Quinn P, editors. Culture media, solutions, and systems in Human ART. Cambridge University Press. P. 142-154

<p><b>CUSTOMER-TECHNICAL SUPPORT</b></p> <p><b>SUPPORT CLIENTS/SUPPORT TECHNIQUE</b></p> <p><b>KUNDENDIENST-TECHNISCHER SUPPORT</b></p> <p><b>ATENCIÓN AL CLIENTE-ASISTENCIA TÉCNICA</b></p> <p><b>ASSISTENZA CLIENTI-SUPPORTO TECNICO</b></p> <p><b>APOIO AO CLIENTE-TÉCNICO</b></p> <p><b>ΥΠΟΣΤΗΡΙΞΗ ΠΛΑΤΑΝΟ-ΤΕΧΝΙΚΗ ΥΠΟΣΤΗΡΙΞΗ</b></p> <p><b>KLANTEN-TECHNISCHE ONDERSTEUNING</b></p>	<p>FertiPro NV</p> <p>Industriepark Noord 32</p> <p>8730 Beernem / Belgium</p> <p>Tel +32 (0)50 79 18 05</p> <p>Fax +32 (0)50 79 17 99</p> <p>URL: <a href="http://www.fertipro.com">www.fertipro.com</a></p> <p>E-mail: <a href="mailto:info@fertipro.com">info@fertipro.com</a></p>	<p><b>CE</b></p> <p><b>2797</b></p>
--	---	-------------------------------------

Notes:

- Test tube culture is now rare, but for those still using this method, place 1ml of FertiCult™ IVF medium in each tube and then follow the same procedure as for Falcon or Nunc dishes.
- Each laboratory should consult its own validated procedures, optimized for its individual medical program.

***Note on optimal pH, co2 and atmospheric pressure***  
For optimal embryo culture, FertiCult™ IVF medium should be used at a pH of 7.20-7.35. Different factors like the CO2 concentration in the incubator and atmospheric pressure (which decreases at higher altitudes) have an effect on the pH after equilibration in the incubator<sup>1</sup>. Therefore, we strongly advise to measure the pH under culture conditions with 6% CO2 in the incubator in order to determine the CO2 concentration settings of the incubator which result in an optimal pH of 7.20-7.35.

### STORAGE/DISPOSAL INSTRUCTIONS

- Store FertiCult™ IVF medium without gentamicin between 2-25 °C. Once opened: store between 2-8 °C.
- Store FertiCult™ IVF medium with gentamicin between 2-8 °C.
- The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8 °C.
- Keep away from sunlight.
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

### PRECAUTIONS

- Aseptic technique should be used to avoid possible contamination, even when the products contain gentamicin.
- All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HbsAg and negative for Anti-HIV-1/2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.
- Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes. Therefore, handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

### SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SSCP)

The SSCP for FertiCult™ IVF medium describes safety and performance characteristics for the media and is available on the website of FertiPro NV ([www.fertipro.com](http://www.fertipro.com)) or by downloading the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

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