FertiCult™ IVF medium

INDICATIONS FOR USE

—

FertiCult™ IVF medium is intended for washing and holding of human oocytes, performing fertilization by IVF or ICSI (until 2PN).

For professional use only.

GENERAL INFORMATION

—

The medium is complete and needs no further additives. As with all IVF media, FertiCult™ IVF medium has to be **pre-incubated in a CO2 incubator with 5-6% CO2 for at least 5 hours, but ideally, dishes should be prepared the day before use and incubated overnight** (with lid opened) to obtain an optimal pH for oocyte washing/holding.

COMPOSITION

FertiCult™ IVF medium is a ready-to-use bicarbonate-buffered balanced salt solution, supplemented with glucose (5.55mM), lactate, pyruvate, and human serum albumin (4.0g/l; medicinal substance derived from human blood plasma). The product is also available with phenol red and/ or gentamicin (10mg/l medicinal substance).

PRODUCT SPECIFICATIONS

- Chemical composition
- pH: 7.20 – 7.50 (37 °C – 6% CO2)
- Osmolality: 270 - 290 mOsm/kg
- Sterility test by the current Ph. Eur. 2.6.1./ USP <71>: No growth
- Endotoxin test by Limulus Ameobocyte Lysate (LAL) methodology (USP <85>): < 0.25EU/ml
- Mouse Embryo Assay (% blastocysts after 96h after IVF in test medium): ≥ 80%
- Use of Ph Eur or USP grade raw materials if applicable Certificate of analysis and MSDS are available upon request or can be downloaded from our website (www.fertipro.com).

STERILE | A

Document ID.: FP09 I07 R01.F.1
Update: 01/12/2021

MATERIAL INCLUDE/ MATERIEL INCLUS/ IM LIEFERUMFANG ENTHALTENE MATERIALIEN/ MATERIAL INCLUDING/ MATERIALI INCLUSI/ MATERIAL INCLUIDO/ ΥΑΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/ MATERIAAL AANWEZIG
<div style="text-align: right;"> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Cataloqe number</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Referenqe catalogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Número de catálogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Numero di catalogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Aplikaos katalogonou</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Catalogosnummer</div> </div>
FertiCult™ IVF medium
FEUJ020 5 x 20ml FertiCult™ IVF medium
FEUJ050 5 x 50ml FertiCult™ IVF medium
FEUJ100 3 x 100ml FertiCult™ IVF medium
FertiCult™ IVF medium with phenol red
FEUJ020PHR 5 x 20ml FertiCult™ IVF medium with phenol red
FEUJ050PHR 5 x 50ml FertiCult™ IVF medium with phenol red
FEUJ100PHR 3 x 100ml FertiCult™ IVF medium with phenol red
FertiCult™ IVF medium with phenol red and gentamicin
FEUJ020PHR_G 5 x 20ml FertiCult™ IVF medium with phenol red and gentamicin
FEUJ050PHR_G 5 x 50ml FertiCult™ IVF medium with phenol red and gentamicin
FEUJ100PHR_G 3 x 100ml FertiCult™ IVF medium with phenol red and gentamicin

<div style="text-align: right;"> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Cataloqe number</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Referenqe catalogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Número de catálogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Numero di catalogo</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Aplikaos katalogonou</div> <div style="writing-mode: vertical-rl; transform: rotate(180deg);">Catalogosnummer</div> </div>
FertiCult™ IVF medium
FEUJ020 5 x 20ml FertiCult™ IVF medium
FEUJ050 5 x 50ml FertiCult™ IVF medium
FEUJ100 3 x 100ml FertiCult™ IVF medium
FertiCult™ IVF medium with phenol red
FEUJ020PHR 5 x 20ml FertiCult™ IVF medium with phenol red
FEUJ050PHR 5 x 50ml FertiCult™ IVF medium with phenol red
FEUJ100PHR 3 x 100ml FertiCult™ IVF medium with phenol red
FertiCult™ IVF medium with phenol red and gentamicin
FEUJ020PHR_G 5 x 20ml FertiCult™ IVF medium with phenol red and gentamicin
FEUJ050PHR_G 5 x 50ml FertiCult™ IVF medium with phenol red and gentamicin
FEUJ100PHR_G 3 x 100ml FertiCult™ IVF medium with phenol red and gentamicin

Important: Read the note on optimal pH, CO2 and atmospheric pressure below carefully.

When pre-equilibration is performed in culture dishes, follow the instructions below:

- For micro-droplets: 6-8 droplets of 25-250µl of FertiCult™ IVF medium may be dispensed along the 6cm culture dish/well. To maintain pH, temperature and osmolality, make sure that the droplets are completely covered with oil (e.g. FertiCult™ Mineral Oil, FertiPro NV). 1-5 oocytes are usually placed per micro-droplet.
- Alternatively, the medium can also be used for fertilization in an open system such as with the Falcon / Nunc dishes. About 1ml of medium is placed in each well. Additional medium is placed in the reservoir surrounding the wells. This helps to maintain humidity as well as providing medium which can be used for washing of the oocytes. Up to 5 oocytes per dish/well may be cultured together. In the open system, paraffin oil is not necessary, but sometimes a 1ml layer of mineral oil may be added over the medium in the inner well. When using such an open system the incubator must be humidified.

Washing/holding of oocytes and fertilization by IVF or ICSI
After equilibration of the medium, the prepared dishes are ready for use. During oocyte collection, oocytes are identified in the follicular aspirates and then washed before placing into the wells or droplets in the dish. After oocyte collection, oocytes will be checked and may be washed, before being placed into fresh droplets/wells for further culture. Generally, within 6 hours of oocyte collection, oocytes will be inseminated by IVF or ICSI with prepared sperm and then left overnight in the incubator. The presence of 2 pro-nuclei (2PN) is checked 16-20 hrs after fertilization. Once embryos have been identified by the presence of 2PN, they are placed into fresh dishes containing a fresh embryo culture medium (e.g GAIN™ Medium, FertiPro NV) for further embryo culture and embryo transfer.

<p>FertiPro NV Industriepark Noord 32 87330 Beernem / Belgium Tel +32 (0)50 79 18 05 Fax +32 (0)50 79 17 99 URL: www.fertipro.com E-mail: info@fertipro.com</p>	<p>CE 2797</p>
--	------------------------------

Notes:

- Test tube culture is now rare, but for those still using this method, place 1ml of FertiCult™ IVF medium in each tube and then follow the same procedure as for Falcon or Nunc dishes.
- Each laboratory should consult its own validated procedures, optimized for its individual medical program.

Note on optimal pH, co2 and atmopheric pressure
For optimal embryo culture, FertiCult™ IVF medium should be used at a pH of 7.20-7.35. Different factors like the CO2 concentration in the incubator and atmospheric pressure (which decreases at higher altitudes) have an effect on the pH after equilibration in the incubator¹. Therefore, we strongly advise to measure the pH under culture conditions with 6% CO2 in the incubator in order to determine the CO2 concentration settings of the incubator which result in an optimal pH of 7.20-7.35.

STORAGE/DISPOSAL INSTRUCTIONS

- Store FertiCult™ IVF medium without gentamicin between 2-25 °C. Once opened: store between 2-8 °C.
- Store FertiCult™ IVF medium with gentamicin between 2-8 °C.
- The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8 °C.
- Keep away from sunlight.
- The products are stable after transport (max. 5 days) at elevated temperatures (≤ 37 °C).
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

PRECAUTIONS

- Aseptic technique should be used to avoid possible contamination, even when the products contain gentamicin.
- All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.
- Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/ removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes. Therefore, handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

WARNINGS BEFORE USE

- Do not use the product if:
 - it becomes discoloured (if medium contains phenol red), cloudy, or shows any evidence of microbial contamination;
 - seal of the container is opened or defect when the product is delivered;
 - expiry date has been exceeded.
- Do not freeze before use.
- Do not re-sterilize after opening.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple opening/ warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use

METHOD

MEDIUM PRE-EQUILIBRATION / DISH PREPARATION

Incubate the medium (in culture dishes or flasks with loosened screw tops) in a CO2 incubator at 37 °C and 5-6% CO2. Ideally the medium should be pre-incubated overnight before use (or at least 5 hours).

Important: Read the note on optimal pH, CO2 and atmospheric pressure below carefully.

When pre-equilibration is performed in culture dishes, follow the instructions below:

- For micro-droplets: 6-8 droplets of 25-250µl of FertiCult™ IVF medium may be dispensed along the 6cm culture dish/well. To maintain pH, temperature and osmolality, make sure that the droplets are completely covered with oil (e.g. FertiCult™ Mineral Oil, FertiPro NV). 1-5 oocytes are usually placed per micro-droplet.
- Alternatively, the medium can also be used for fertilization in an open system such as with the Falcon / Nunc dishes. About 1ml of medium is placed in each well. Additional medium is placed in the reservoir surrounding the wells. This helps to maintain humidity as well as providing medium which can be used for washing of the oocytes. Up to 5 oocytes per dish/well may be cultured together. In the open system, paraffin oil is not necessary, but sometimes a 1ml layer of mineral oil may be added over the medium in the inner well. When using such an open system the incubator must be humidified.

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For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

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