
Sil-Select Stock™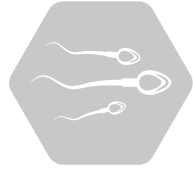

Available languages

EN	page	2
FR	page	3
DE	Seite	4
ES	página	5
IT	pagina	6
PT	página	7
GR	σελίδα	8
NL	pagina	9
BG	страница	10
LT	puslapis	11
NO	side	12
TR	Sayfa	13
HU	oldal	14
DK	side	15

Sil-Select Stock™

INTENDED USE AND COMPOSITION

Sil-Select Stock is a stock solution for semen preparation. It is an isotonic gradient for semen preparation with a density of approximately 1.12 g/ml. Sil-Select Stock consists of silane-coated colloidal silica particles suspended in HEPES-buffered EBSS (Earle's balanced salt solution) and some product codes are supplemented with 10mg/l gentamicin (medicinal substance). Sil-Select Stock can be used for semen preparation for Intra Uterine Insemination (IUI), In Vitro Fertilization (IVF) and IntraCytoplasmic Sperm Injection (ICSI).

For professional use only.

PRODUCT SPECIFICATIONS

- Chemical composition
- pH criteria: 7.20-7.90 (Release criteria: 7.20-7.60)
- Osmolality: 300-330 mOsm/kg
- Density: 1.1150-1.1250 g/ml
- Endotoxin test by Limulus Amebocyte Lysate methodology (USP <85>): < 0.5EU/ml
- Sterility test according to the current Ph. Eur. 2.6.1/USP <71>: No growth
- Human sperm survival assay (% motility compared with control after 4 hours): ≥ 80%
- Human sperm survival assay (% motility compared with control after 24 hours): ≥ 75%
- Not MEA tested
- Use of Ph Eur or USP grade products if applicable
- A certificate of analysis and MSDS are available upon request or can be downloaded from our website (www.fertipro.com)

CALCULATIONS OF G-FORCES

The g-force of your centrifuge can be calculated using this formula:

$$g = 1.118 \times r \times \text{rpm}^2$$

$$\text{rpm} = \text{Square root} \left\{ g / (1.118 \times r) \right\}$$

r = radius of centrifuge in mm
 rpm = rotations per minute / 1000

Example 1

r = 100 mm
 rpm = 1800 rotations per minute
 g = 1.118 x 100 x 3.24 = 362 g

Example 2

r = 100 mm
 g = 350g
 rpm = SQR (350 / (1.118 x 100)) = 1.77
 rpm = 1770 rotations per minute

WARNINGS BEFORE USE

- Do not use the product if:
 - it shows any evidence of microbial contamination
 - seal of the container is opened or defect when the product is delivered
 - expiry date has been exceeded
- Do not freeze before use.
- Do not re-sterilize after opening.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use

METHOD

Importantly: each laboratory should consult its own optimized and validated laboratory procedures. Further reference is made to our website for demonstration video's.

INSTRUCTIONS FOR PREPARATION OF GRADIENTS

Mix the density gradient bottles by 5 bottle inversions before use.

We advise to prepare a dual gradient system (45% – 90% or 40% – 80%) starting from Sil-Select Stock. If preferred a multi-layer can be used as well (e.g. 45% – 70% – 90%). To prepare a 90% gradient, mix 1 part FertiCult Flushing medium (or Sil-Select Plus Washing/Insemination medium, FertiPro NV) with 9 parts Sil-Select Stock. A 45% gradient is prepared by mixing 5.5 parts FertiCult Flushing medium (or Sil-Select Plus Washing/Insemination medium, FertiPro NV) with 4.5 parts Sil-Select Stock. Alternatively any HEPES-buffered EBSS-based medium can be used for the preparation of the gradients.

Note: Gradients should be prepared and repacked under sterile conditions (e.g. LAF bench ISO Class 5). For optimal results, prepare the gradients maximum 24 hours prior to use, store at 2-8 °C and warm gradients to room temperature or 37 °C one hour before use. Mix well after diluting the Sil-Select Stock solution.

INSTRUCTIONS FOR SPERM SELECTION WITH FRESH SEMEN SAMPLES: EXAMPLE USING A 45%-90% GRADIENT SYSTEM (BUT OTHER GRADIENTS ARE POSSIBLE)

- 1 Transfer 2.5ml of the prepared 45% gradient into a sterile disposable centrifuge tube.
- 2 Using a syringe with needle, place 2.5ml of the prepared 90% gradient under the prepared 45% gradient. Take care that the two layers are distinctly separated. This is done by placing the tip of the needle on the bottom of the test tube and slowly dispensing the 90% gradient. This two layer gradient is stable for up to two hours.
- 3 Gently place up to 2.5ml of liquefied semen onto the 45% gradient using a transfer pipette or syringe. Do not use a higher volume than the volume of the individual gradient layers or more than 10⁹ cells.
- 4 Centrifuge for 15 to 18 minutes at 350g to 400g. When this centrifugation is completed you may not be able to visibly see a pellet. If so, it is essential to continue the procedure with a second centrifugation of 3 to 5 minutes.
- 5 Remove supernatant down to the pellet.
- 6 Using a syringe, add 2-3ml of sperm washing medium (e.g. FertiCult Flushing medium or Sil-Select Plus Washing/Insemination medium, FertiPro NV) and resuspend the pellet.
- 7 Centrifuge for 8 to 10 minutes at 300g. Higher sperm concentration will require the maximum 10 minutes centrifugation to ensure a complete and thorough sperm wash.
- 8 Remove supernatant down to the pellet and repeat steps 6 and 7.
- 9 Remove supernatant and replace with a suitable volume of appropriate medium.

If samples do not liquefy and therefore do not pass through the layers, increasing the centrifugal force up to, but no more than, 500g will help to separate the sperm.

INSTRUCTIONS FOR SPERM SELECTION WITH FROZEN SEMEN SAMPLES: EXAMPLE USING A 45%-90% GRADIENT SYSTEM (BUT OTHER GRADIENTS ARE POSSIBLE)

- 1 Transfer 1ml of the prepared 45% gradient into a sterile disposable centrifuge tube.
- 2 Using a syringe with needle, place 1ml of the prepared 90% gradient under the prepared 45% gradient. Take care that the two layers are distinctly separated. This is done by placing the tip of the needle on the bottom of the test tube and slowly dispensing the 90% gradient. This two-layer gradient is stable for up to two hours.
- 3 Gently place the thawed semen sample onto the 45% gradient using a transfer pipette or syringe (0.5ml maximum).
- 4 Centrifuge for 15-20 minutes at 350g.
- 5 Remove supernatant down to no less than the 0.5ml mark above the pellet.
- 6 Using a syringe, add 2-3ml of sperm washing medium (e.g. FertiCult Flushing medium or Sil-Select Plus Washing/Insemination medium, FertiPro NV) and resuspend the pellet.
- 7 Centrifuge for 8 to 10 minutes at 300g.
- 8 Remove supernatant down to the pellet and repeat steps 6 and 7.
- 9 Remove supernatant and replace with a suitable volume of appropriate medium.

STORAGE/DISPOSAL INSTRUCTIONS

- Store products without gentamicin between 2-25 °C. Once opened: store between 2-8 °C.
- Store products with gentamicin between 2-8 °C.
- The products can be used up to 7 days after opening when sterile conditions are maintained and the products are stored at 2-8 °C.
- Keep away from (sun)light
- The products are stable after transport (max. 5 days) at elevated temperatures (≤ 37 °C)
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

PRECAUTIONS

- Aseptic technique should be used to avoid possible contamination, even when the products contain gentamicin.
- All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SCCP)

The SSCP for Sil-Select Stock describes safety and performance characteristics for the media and is available on the website of FertiPro NV (www.fertipro.com) or by scanning the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

SYMBOLS GLOSSARY

Symbols as defined in ISO 15223-1

REF Catalogue number

LOT Batch code

Use-by date

Manufacturer

Keep away from sunlight

MD Medical device

Consult instructions for use

Cautions

Date of manufacture

STERILE A Sterilized using aseptic processing techniques

°C Temperature limit

Single sterile barrier system

Single sterile barrier system with protective packaging outside.

Do not use if package is damaged and consult instructions for use.

Symbol as defined in EU Commission MDR 2017/745

CE CE marking by Notified Body 2797
2797

Symbol as defined in 21 CFR 801.15

R ONLY US Federal law restricts this device to sale by or on the order of a physician.

STERILE A

Document ID: FP09 I13_Stock R01 E.4
 Update: 01/12/2021

MATERIAL INCLUDED

Catalogue number

SIS100	1 x 100ml	Sil-Select Stock™
SIS100_32G	1 x 100ml	Sil-Select Stock™
SIS500	1 x 500ml	Sil-Select Stock™
SIS100G	1 x 100ml	Sil-Select Stock™ with gentamicin
SIS100_32G	1 x 100ml	Sil-Select Stock™ with gentamicin
SIS500G	1 x 500ml	Sil-Select Stock™ with gentamicin

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