Sil-Select Stock™

Available languages

EN	page	2
FR	page	3
DE	Seite	4
ES	página	5
IT	pagina	6
PT	página	7
GR	σελίδα	8
NL	pagina	9
BG	страница	10
LT	puslapis	11
NO	side	12
TR	Sayfa	13
HU	oldal	14
DK	side	15

Sil-Select Stock™

GENERAL

Intended use

Sil-Select Stock is a stock solution used as sperm preparation method for intrauterine insemination (IUI), in vitro fertilization (IVF), intracytoplasmic sperm injection (ICSI), and related assisted reproductive technologies (ART).

Indications for use

For use during ART procedures of patients and couples undergoing infertility treatments.

Intended user

The intended users are ART professionals (lab technicians, embryologists, or medical doctors).

Target patient population

The target patient population consists of patients and couples undergoing infertility treatments.

COMPOSITION

Sil-Select Stock is an isotonic gradient with a density of approximately 1.12 g/ml. Sil-Select Stock consists of silane-coated colloidal silica particles suspended in HEPES-buffered EBSS (Earle's balanced salt solution). Some product codes are supplemented with gentamicin (10 mg/l, medicinal substance).

PRODUCT SPECIFICATIONS

- Chemical composition
- pH criteria: 7.20-7.90 (Release criteria: 7.20-7.60)
- Osmolality: 300-330 mOsm/kg
- Density: 1.1150-1.1250 g/ml
- Endotoxin test by Limulus Amebocyte Lysate methodology (USP <85>): < 0.5EU/ml
- Sterility test according to the current Ph. Eur. 2.6.1/USP <71>: No growth
- Human sperm survival assay (% motility compared with control after 4 hours): ≥ 80%
- Human sperm survival assay (% motility compared with control after 24 hours): ≥ 75%
- Not MEA tested
- Use of Ph Eur or USP grade products if applicable
- A certificate of analysis and MSDS are available upon request or can be downloaded from our website (www.fertipro.com)

CALCULATIONS OF G-FORCES

The g-force of your centrifuge can be calculated using this formula:

$$g = 1.118 \times r \times \text{rpm}^2$$

$$\text{rpm} = \text{Square root} [g / (1.118 \times r)]$$

$$r = \text{radius of centrifuge in mm}$$

$$\text{rpm} = \text{rotations per minute} / 1000$$

Example 1

r = 100 mm
rpm = 1800 rotations per minute
g = 1.118 x 100 x 3.24 = 362 g

Example 2

r = 100 mm
g = 350g
rpm = $\sqrt{350 / (1.118 \times 100)}$ = 1.77
rpm = 1770 rotations per minute

WARNINGS BEFORE USE

- Do not use the product if:
 - it shows any evidence of microbial contamination
 - seal of the container is opened or defect when the product is delivered
 - expiry date has been exceeded
- Do not freeze before use.
- Do not re-sterilize after opening.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed

on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/warming cycles of the medium. Discard excess (unused) media.

- Keep in its original packaging until the day of use

METHOD

Importantly: each laboratory should consult its own optimized and validated laboratory procedures. Further reference is made to our website for demonstration video's.

INSTRUCTIONS FOR PREPARATION OF GRADIENTS

Mix the density gradient bottles by 5 bottle inversions before use.

We advise to prepare a dual gradient system (45% - 90% or 40% - 80%) starting from Sil-Select Stock. If preferred a multi-layer can be used as well (e.g. 45% - 70% - 90%). To prepare a 90% gradient, mix 1 part FertiCult Flushing medium (or Sil-Select Plus Washing/Insemination medium, FertiPro NV) with 9 parts Sil-Select Stock. A 45% gradient is prepared by mixing 5.5 parts FertiCult Flushing medium (or Sil-Select Plus Washing/Insemination medium, FertiPro NV) with 4.5 parts Sil-Select Stock. Alternatively any HEPES-buffered EBSS-based medium can be used for the preparation of the gradients.

Note: Gradients should be prepared and repacked under sterile conditions (e.g. LAF bench ISO Class 5). For optimal results, prepare the gradients maximum 24 hours prior to use, store at 2-8 °C and warm gradients to room temperature or 37 °C one hour before use. Mix well after diluting the Sil-Select Stock solution.

INSTRUCTIONS FOR SPERM SELECTION WITH FRESH SEMEN SAMPLES: EXAMPLE USING A 45%-90% GRADIENT SYSTEM (BUT OTHER GRADIENTS ARE POSSIBLE)

1. Transfer 2.5ml of the prepared 45% gradient into a sterile disposable centrifuge tube.
2. Place 2.5ml of the prepared 90% gradient under the prepared 45% gradient. Take care that the two layers are distinctly separated. This is done by placing the tip of the pipette on the bottom of the test tube and slowly dispensing the 90% gradient. This two layer gradient is stable for up to two hours.
3. Gently place up to 2.5ml of liquefied semen onto the 45% gradient using a transfer pipette. Do not use a higher volume than the volume of the individual gradient layers or more than 10⁹ cells.
4. Centrifuge for 15 to 18 minutes at 350g to 400g. When this centrifugation is completed you may not be able to visibly see a pellet. If so, it is essential to continue the procedure with a second centrifugation of 3 to 5 minutes.
5. Remove supernatant down to the pellet.
6. Add 2.5ml of sperm washing medium (e.g. FertiCult Flushing medium or Sil-Select Plus Washing/Insemination medium, FertiPro NV) and resuspend the pellet.
7. Centrifuge for 8 to 10 minutes at 300g. Higher sperm concentration will require the maximum 10 minutes centrifugation to ensure a complete and thorough sperm wash.
8. Remove supernatant down to the pellet and repeat steps 6 and 7.
9. Remove supernatant and replace with a suitable volume of appropriate medium.

If samples do not liquefy and therefore do not pass through the layers, increasing the centrifugal force up to, but no more than, 500g will help to separate the sperm.

INSTRUCTIONS FOR SPERM SELECTION WITH FROZEN SEMEN SAMPLES: EXAMPLE USING A 45%-90% GRADIENT SYSTEM (BUT OTHER GRADIENTS ARE POSSIBLE)

1. Transfer 1ml of the prepared 45% gradient into a sterile disposable centrifuge tube.
2. Place 1ml of the prepared 90% gradient under the prepared 45% gradient. Take care that the two layers are distinctly separated. This is done by placing the tip of the pipette on the bottom of the test tube and slowly dispensing the 90% gradient. This two-layer gradient is stable for up to two hours.
3. Gently place the thawed semen sample onto the 45% gradient using a transfer pipette (0.5 - 1ml).

- 4. Centrifuge for 15-20 minutes at 350g.
- 5. Remove supernatant down to no less than the 0.5ml mark above the pellet.
- 6. Add 2.5ml of sperm washing medium (e.g. FertiCult Flushing medium or Sil-Select Plus Washing/Insemination medium, FertiPro NV) and resuspend the pellet.
- 7. Centrifuge for 8 to 10 minutes at 300g.
- 8. Remove supernatant down to the pellet and repeat steps 6 and 7.
- 9. Remove supernatant and replace with a suitable volume of appropriate medium.

STORAGE/DISPOSAL INSTRUCTIONS

- Store products between 2-8 °C.
- The products can be used up to 7 days after opening when sterile conditions are maintained and the products are stored at 2-8 °C.
- Keep away from (sun)light.
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

PRECAUTIONS

- Aseptic technique should be used to avoid possible contamination, even when the products contain gentamicin.
- All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SSCP)

The SSCP for Sil-Select Stock describes safety and performance characteristics for the media and is available on the website of FertiPro NV (www.fertipro.com) or by scanning the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

SYMBOLS GLOSSARY

Symbols as defined in ISO 15223-1	
	Catalogue number
	Batch code
	Use-by date
	Manufacturer
	Keep away from sunlight
	Medical device
	Consult instructions for use
	Cautions
	Date of manufacture
	Sterilized using aseptic processing techniques
	Temperature limit
	Single sterile barrier system
	Single sterile barrier system with protective packaging outside.
	Do not use if package is damaged and consult instructions for use.
	Do not resterilize
	Contains a medicinal substance

Symbol as defined in EU Commission MDR 2017/745

CE marking by Notified Body 2797

Symbol as defined in 21 CFR 801.15

US Federal law restricts this device to sale by or on the order of a physician.

STERILE A

Document ID: FPO9 I13_Stock R01 E.5
Update: 17/04/2025

MATERIAL INCLUDED

Catalogue number		
SIS100	1 x 100ml	Sil-Select Stock™
SIS100_32	1 x 100ml	Sil-Select Stock™
SIS500	1 x 500ml	Sil-Select Stock™
SIS100_32G	1 x 100ml	Sil-Select Stock™ with gentamicin

CUSTOMER-TECHNICAL SUPPORT

FertiPro NV
Industriepark Noord 32
8730 Beernem / Belgium
Tel +32 (0)50 79 18 05
Fax +32 (0)50 79 17 99
URL: www.fertipro.com
E-mail: info@fertipro.com

Sil-Select Stock™

STERILE A

Document ID: FPO9 I13_Stock R01 E.5
Update: 17/04/2025

MATÉRIEL INCLUS

Référence catalogue		
SIS100	1 x 100ml	Sil-Select Stock™
SIS100_32	1 x 100ml	Sil-Select Stock™
SIS500	1 x 500ml	Sil-Select Stock™
SIS100_32G	1 x 100ml	Sil-Select Stock™ with gentamicin

SUPPORT CLIENTS-SUPPORT TECHNIQUE

FertiPro NV
Industriepark Noord 32
8730 Beernem / Belgium
Tel +32 (0)50 79 18 05
Fax +32 (0)50 79 17 99
URL: www.fertipro.com
E-mail: info@fertipro.com

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