SpermFreeze Box

Document ID: FP09 I122 R01 B.1
Update: 27/01/2026

MATERIAL INCLUDED/
MATÉRIEL INCLUS/
IM LIEFERUMFANG ENTHALTENE MATERIALIEN/
MATERIAL INCLUIDO/
MATERIALI INCLUSI/
MATERIAL INCLUIDO/
ΥΛΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/
MATERIAAL AANWEZIG

SPF_BOX	SpermFreeze Box and floater
SPF_FLOATER_STR	Floater
SPF_VIAL	Insert

Catalogue number
Référence catalogue
Bestelnummer
Número de catálogo
Numero di catalogo
Αριθμός καταλόγου
Catalogusnummer

SpermFreeze Box

GENERAL

Intended use
The SpermFreeze Box is a box to standardize the sperm freezing procedure. It offers a standardized height of the sperm freezing straw or cryovial in the liquid nitrogen vapour phase.

Indications for use
For use during ART procedures of patients and couples undergoing infertility treatments.

Intended user
The intended users are ART professionals (lab technicians, embryologists, or medical doctors).

Target patient population
The target patient population consists of patients and couples undergoing infertility treatments.

COMPOSITION

Box, floater and insert in expanded poly propylene (EPP)

MATERIAL NOT INCLUDED BUT NEEDED

- Sperm freezing medium: SpermFreeze / SpermFreeze SSP (FertiPro NV)
- Sperm freezing straws or cryovials (2 ml)
- Freezing tank with liquid nitrogen

METHOD

- See figure 1
- Freezing sperm samples in straws:**
(needed: SpermFreeze Box and floater)
- Fill the box with 1-2 liter of liquid nitrogen
 - Place your sealed straws (maximum 15) on the floater
 - Deposit the floater on the liquid nitrogen
 - Close the box by placing the lid loosely on the box
 - Leave for (at least) 15 minutes
 - Plunge the straws in the liquid nitrogen
 - Store the straws in your nitrogen storage container for long term preservation

- Freezing sperm samples in cryovials:**
(needed: SpermFreeze Box, floater and insert)
- Note:** the following method is validated for 2 ml cryovials. For cryovials of other volumes, each laboratory should consult its own validated procedures.
- Fill the box with 1-2 liter of liquid nitrogen.
 - Place the insert on the floater. Place the closed cryovials (max 8) vertically through the insert (only cap is visible above the insert).
 - Deposit the floater with insert in the liquid nitrogen.
 - Close the box by placing the lid loosely on the box
 - Leave for (at least) 15 minutes.
 - Plunge the cryovials in the liquid nitrogen.
 - Store the cryovials in your nitrogen storage container for long term preservation.

PRE-USE CHECKS

Before each use, check if the SpermFreeze Box, floater and/or insert shows any damage. Do not use if defects are noticed.

STORAGE INSTRUCTIONS

- Empty the SpermFreeze Box after each use
- The SpermFreeze Box, floater and/or insert can be used multiple times, is washable (dishwasher) and disinfectable (using disinfectants standardly used in an ART-lab)

WARNING AND PRECAUTIONS

Wear personal protective equipment during use and manipulations of liquid nitrogen to protect yourself

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