# SpermFreeze™

## Available languages

EN	<a href="#">page</a>	2
FR	<a href="#">page</a>	3
DE	<a href="#">Seite</a>	4
ES	<a href="#">página</a>	5
IT	<a href="#">pagina</a>	6
PT	<a href="#">página</a>	7
GR	<a href="#">σελίδα</a>	8
NL	<a href="#">pagina</a>	9
BG	<a href="#">страница</a>	10
NO	<a href="#">side</a>	11
RO	<a href="#">pagina</a>	12
SK	<a href="#">stran</a>	13
TR	<a href="#">Sayfa</a>	14
HU	<a href="#">oldal</a>	15
DK	<a href="#">side</a>	16

## INDICATIONS FOR USE

SpermFreeze™ is a medium for cryopreservation of human sperm for further use in Assisted Reproductive Technologies. SpermFreeze™ can also be used for the cryopreservation of sperm from testicular biopsies (TESE).

*For professional use only.*

## COMPOSITION

SpermFreeze™ is a ready-to-use HEPES buffered cryopreservation medium which also contains physiologic salts, glycine, glucose, lactate and the cryoprotectants glycerol (15%), sucrose and Human Serum Albumin (4.0g/l; medicinal substance derived from human blood plasma) to protect the sperm from damage during the freezing procedure.

## ACCESSORIES

The following device can be used in combination with SpermFreeze™ to further standardize the cryopreservation process:

- SpermFreeze Box, product code SPF\_BOX (FertiPro NV)

## MATERIAL NOT INCLUDED BUT NEEDED

- Sperm freezing straws (e.g. CBS™ high security sperm straws) or cryo tubes
- Freezing tank with liquid nitrogen

## PRODUCT SPECIFICATIONS

- Chemical composition
- pH between 7.20 – 7.90 (Release criteria: 7.20 – 7.60)
- Sterility test by the current Ph. Eur. 2.6.1 / USP <71>: No growth
- Endotoxins (USP <85>): < 0.25EU/ml
- Human sperm survival assay (% motility compared with control after 4 hours): ≥ 80%
- Not MEA tested
- Use of Ph Eur or USP grade products if applicable
- Certificate of analysis and MSDS are available upon request or can be downloaded from our website (www.fertipro.com).

## WARNINGS BEFORE USE

- Do not use the product if:
  - it becomes cloudy, or shows any evidence of microbial contamination;
  - seal of the container is opened or defect when the product is delivered;
  - expiry date has been exceeded.
- Do not freeze before use.
- Do not re-sterilize after opening.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use

## STORAGE/DISPOSAL INSTRUCTIONS

- Store between 2-8 °C.
- Keep away from (sun)light.
- The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8 °C.
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

## METHOD

Ensure all media are well mixed and at room temperature before use.

### Before freezing

Ideally, cryopreservation is performed on native semen samples. In case of very low sperm concentrations it is advisable to concentrate the sperm before freezing. In case of very high sperm concentration, sperm can be diluted with FertiCult™ Flushing medium before freezing, if preferred.

### Freezing

- 1 Allow the semen to liquefy at room temperature for 30 minutes.
- 2 Mix 1ml of sperm with 0.7-1ml of SpermFreeze™. Importantly: add the medium in drops while gently swirling. Caution: to avoid cold-shock, make sure SpermFreeze™ is at room temperature.
- 3 Leave the mixture for 10 minutes at room temperature for equilibration.
- 4 Suck the sample/medium mixture into the freezing straws, leaving approximately 1.5cm of air at the end of the straw.
- 5 Make sure that the outer surface of the straw is clean (no sperm) and seal the straw.
- 6 If applicable: dry off individually with a dust free cloth.
- 7 Shake to move the air-bubble to the center of the straw
- 8 Place the straws horizontally (e.g. on a styrofoam board) in a liquid nitrogen bath to allow for freezing in vapor phase. Leave for (at least) 15 minutes.
- 9 Transfer straws quickly into liquid nitrogen and store at -196 °C.

When using the SpermFreeze Box as accessory, step 8 and 9 in the freezing process are adapted as follows:

- Fill the box with 1-2 liter of liquid nitrogen
- Place your sealed straws (maximum 15) on the floater
- Deposit the floater on the liquid nitrogen
- Close the box by placing the lid loosely on the box
- Leave for (at least) 15 minutes
- Plunge the straws in the liquid nitrogen
- Store them in your nitrogen storage container for long term preservation

### Thawing

- 1 Remove as many straws as required from the liquid nitrogen.
- 2 Place the straws in tap water for 5 minutes (room temperature or 37 °C).
- 3 Cut off the end of the straw, place the open end inside a container (e.g. a test tube) and tap the straw against the side of the container to allow complete evacuation of the mixture.
- 4 Dilute the concentrated sperm in a suitable insemination medium (e.g. FertiCult™ Flushing medium) (at least 3ml per 0.5ml semen) and mix thoroughly.
- 5 Centrifuge during 15 minutes at 300-350g.
- 6 Resuspend pellet in a suitable insemination medium (e.g. FertiCult™ Flushing medium) and assess recovery.

### Notes:

- 1 If necessary, use sperm preparation techniques after thawing the semen to eliminate dead sperm cells and debris. Dilute the concentrated sperm in a suitable insemination medium (e.g. FertiCult™ Flushing medium).
- 2 For additional details on the method, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for its individual medical program.

## PRECAUTIONS

- All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HbsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.
- Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes. Therefore, handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Aseptic techniques should be used to avoid possible contamination. SpermFreeze™ does not contain any antibiotics.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV and, if applicable, the Competent Authority of the EU Member State in which the user and/ or patient is established.

## SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SCP)

Although damaging effects on sperm motility, vitality, DNA fragmentation and fertilizing potential upon thawing are inherent to the cryopreservation process, these remain within acceptable limits for SpermFreeze™ as reported in scientific literature. The SSCP for SpermFreeze™ medium describes safety and performance characteristics for the media and is available on the website of FertiPro NV (www.fertipro.com) or by using the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

## SYMBOLS GLOSSARY

Symbols as defined in ISO 15223-1	
	Catalogue number
	Batch code
	Use-by date
	Manufacturer
	Keep away from sunlight
	Medical device
	Consult instructions for use
	Cautions
	Date of manufacture
	Sterilized using aseptic processing techniques
	Temperature limit
	Single sterile barrier system
	Single sterile barrier system with protective packaging outside.
	Do not use if package is damaged and consult instructions for use.
	Do not resterilize
	Contains a medicinal substance
	Contains human blood or plasma derivatives

Symbol as defined in EU Commission MDR 2017/745

CE marking by Notified Body 2797

Symbol as defined in 21 CFR 801.15

US Federal law restricts this device to sale by or on the order of a physician.

Document ID: FP09 I11 R01 E.5  
Update: 14/06/2024

## MATERIAL INCLUDED

Catalogue number	
SPF05	25 x 5ml SpermFreeze™
SPF	5 x 20ml SpermFreeze™

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