FertiCult™ Flushing medium

STERILE A
--


Document ID: FP09 I08 R01 D.6
 Update: 04/04/2024

MATERIAL INCLUDED/
MATÉRIEL INCLUS/
IM LIEFERUMFANG ENTHALTENE MATERIALIEN/
MATERIAL INCLUIDO/
MATERIALI INCLUS/
MATERIAL INCLUIDO/
ΥΑΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/
MATERIAAL AANWEEZIG

Catalogue number Référence catalogue Bestelnummer Número de catálogo Número do catálogo Αριθμός καταλόγου Catalogusnummer	
FLUSH020	5 x 20ml FertiCult™ Flushing medium
FLUSH050	5 x 50ml FertiCult™ Flushing medium
FLUSH100	3 x 100ml FertiCult™ Flushing medium
FLUSH_002_PHR	1 x 2ml FertiCult™ Flushing medium with phenol red
FLUSH020PHR	5 x 20ml FertiCult™ Flushing medium with phenol red
FLUSH050PHR	5 x 50ml FertiCult™ Flushing medium with phenol red
FLUSH100PHR	3 x 100ml FertiCult™ Flushing medium with phenol red
FLUSH500PHR	1 x 500ml FertiCult™ Flushing medium with phenol red
FLUSH020PRG	5 x 20ml FertiCult™ Flushing medium with phenol red and gentamicin
FLUSH050PRG	5 x 50ml FertiCult™ Flushing medium with phenol red and gentamicin
FLUSH100PHR_G	3 x 100ml FertiCult™ Flushing medium with phenol red and gentamicin
FLUSH500PHR_G	1 x 500ml FertiCult™ Flushing medium with phenol red and gentamicin
WASH005	25 x 5ml Sil-Select Plus™ Sperm Washing/Insemination medium
WASH010	25x 10ml Sil-Select Plus™ Sperm Washing/Insemination medium
WASH020	5 x 20ml Sil-Select Plus™ Sperm Washing/Insemination medium
WASH050	1 x 50ml Sil-Select Plus™ Sperm Washing/Insemination medium
WASH100	1 x 100ml Sil-Select Plus™ Sperm Washing/Insemination medium
WASH100G	1 x 100ml Sil-Select Plus™ Sperm Washing/Insemination medium with gentamicin

CUSTOMER-TECHNICAL SUPPORT
SUPPORT CLIENTS/SUPPORT TECHNIQUE
KUNDENDIENST-TECHNISCHER SUPPORT
ATENCIÓN AL CLIENTE/ASISTENCIA TÉCNICA
ASSISTENZA CLIENTI/SUPPORTO TECNICO
ΑΠΟΙΟ ΑΝ ΚΛΙΕΝΤΕ-ΤΕΧΝΙΚΟ
ΥΠΟΣΤΗΡΙΞΗ ΠΕΛΑΤΩΝ-ΤΕΧΝΙΚΗ ΥΠΟΣΤΗΡΙΞΗ
KLANTEN-TECHNISCH E ONDERSTEUNING

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FertiPro NV
 Industriepark Noord 32
 8730 Beernem / Belgium
 Tel +32 (0)50 79 18 05
 Fax +32 (0)50 79 17 99
 URL: www.fertipro.com
 E-mail: info@fertipro.com

FertiCult™ Flushing medium

INDICATIONS FOR USE

GENERAL INFORMATION

FertiCult™ Flushing medium is intended for *in vitro* procedures involving human gametes (sperm and oocytes), including washing of gametes, sperm swim-up procedures, intra-uterine insemination (IUI) of the spermatozoa and sperm injection during intracytoplasmic sperm injection (ICSI). FertiCult™ Flushing medium can also be used in human embryo washing and holding, and for embryo transfer (ET) in the uterine cavity.

For professional use only.

COMPOSITION

The medium is complete and needs no further additives. The medium contains HEPES; **no CO₂ incubation is required.** FertiCult™ Flushing medium does not contain heparin, thus for oocyte pick-up, use FertiCult™ Aspiration medium.

METHOD

FertiCult™ Flushing medium is a ready-to-use HEPES-buffered medium which also contains bicarbonate, physiologic salts, glucose, lactate, pyruvate and human serum albumin (4.0g/L, medicinal substance derived from human blood plasma).

The product is also available with phenol red and/or gentamicin (10mg/L, medicinal substance).

PRODUCT SPECIFICATIONS

- Chemical composition
- pH release criteria: 7.30-7.60, pH stability criteria: 7.30-7.90
- Osmolality: 270-290 mOsm/kg
- Endotoxin test by Limulus Amebocyte Lystate methodology (USP <85>): < 0.25EU/ml
- Sterility test by the current Ph. Eur. 2.6.1/ USP <71>: No growth
- One-cell mouse embryo assay (% blastocysts after 96 hours, exposure time to test medium: 60 minutes): ≥ 80%
- Human sperm survival assay (% motility compared with control after 24 hours): ≥ 80%
- Use of Ph Eur or USP grade products if available
- A certificate of analysis and MSDs are applicable upon request or can be downloaded from our website (www.fertipro.com)

WARNINGS BEFORE USE

- Do not use the product if:
 - it becomes discoloured (if medium contains phenol red), cloudy, or shows any evidence of microbial contamination;
 - seal of the container is opened or defect when the product is delivered;
 - expiry date has been exceeded.
- Do not freeze before use.
- Do not re-sterilize after opening.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use

METHOD

As this medium does contain HEPES, the medium should not be placed in a CO₂ incubator prior to use. Pre-incubation in incubator (without CO₂) or heating plate is required to ensure that the temperature of the medium is 37 °C at use. Make sure that the lid is closed when bottles are placed in an incubator with CO₂ as this will lower the pH to 6.90-6.95.

Each laboratory should consult its own validated procedures, optimized for its individual medical program. When performing IUI or ET, follow the instructions of the specific catheters used. In addition, some additional information on the following procedures is provided below.

Washing of spermatozoa (suggested procedure)

The washing of spermatozoa can be done at room temperature or at 37 °C.

- Add 5ml FertiCult™ Flushing medium to the native semen sample and mix. Centrifuge for 5-10 minutes at approximately 300g.
- Remove supernatant and leave about 0.5ml of semen in the centrifuge tube.
- Add 5ml FertiCult™ Flushing medium to the test tube. Mix the solution gently until the pellet is completely dissolved.
- Centrifuge again for 5-10 minutes at 300g.
- Resuspend in a suitable volume of FertiCult™ Flushing medium.

Swim-up procedure (according to WHO, 2021)

- Gently layer 1.5ml FertiCult™ Flushing medium over 1ml of semen in a round bottom centrifuge

- tube.
- Incline the tube at an angle of 45 ° and incubate for 1 hour at 37 °C.
- Gently turn to the upright position and remove the uppermost 1ml.
- Dilute this aliquot of motile cells with 2-5ml FertiCult™ Flushing medium. Centrifuge for 5-10 minutes at 300g, remove supernatant and finally resuspend in 0.5ml of FertiCult™ Flushing medium.

STORAGE/DISPOSAL INSTRUCTIONS

- Store FertiCult™ Flushing medium without gentamicin between 2-25 °C. Once opened: store between 2-8 °C.
- Store FertiCult™ Flushing medium with gentamicin between 2-8 °C.
- The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8 °C.
- Keep away from (sun)light.
- The devices need to be disposed in accordance with local regulations for disposal of medical devices.

PRECAUTIONS

- Aseptic technique should be used to avoid possible contamination, even when the products contain gentamicin.
- All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HbsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.
- Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes. Therefore, handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV, and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

- Always wear protective clothing when handling specimens.
- Any serious incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to FertiPro NV, and, if applicable, to the competent authority of the EU Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SSCP)

The SSCP for FertiCult™ Flushing medium describes safety and performance characteristics for the media and is available on the website of FertiPro NV (www.fertipro.com) or by downloading the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

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