# FertiCult™ IVF medium

STERILE A

Document ID: FPO9 I07 R01 F2  
Update: 07/12/2023

MATERIAL INCLUDED/  
MATERIEL INCLUS/  
IM LIEFERUMFANG ENTHALTENE MATERIALIEN/  
MATERIAL INCLUIDO/  
MATERIALI INCLUSI/  
MATERIAL INCLUIDO/  
ΥΛΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/  
MATERIAAL AANWEZIG

Catalogue number  
Référence catalogue  
Bestellnummer  
Número de catálogo  
Número di catalogo  
Αριθμός καταλόγου  
Catalogus nummer

FertiCult™ IVF medium  
FEU020 5 x 20ml FertiCult™ IVF medium  
FEU050 5 x 50ml FertiCult™ IVF medium  
FEU100 3 x 100ml FertiCult™ IVF medium  
  
FertiCult™ IVF medium with phenol red  
FEUCU02OPHR 5 x 20ml FertiCult™ IVF medium with phenol red  
FEUCU05OPHR 5 x 50ml FertiCult™ IVF medium with phenol red  
FEUCU100OPHR 3 x 100ml FertiCult™ IVF medium with phenol red  
  
FertiCult™ IVF medium with phenol red and gentamicin  
FEUCU02OPHR\_G 5 x 20ml FertiCult™ IVF medium with phenol red and gentamicin  
FEUCU05OPHR\_G 5 x 50ml FertiCult™ IVF medium with phenol red and gentamicin  
FEUCU100OPHR\_G 3 x 100ml FertiCult™ IVF medium with phenol red and gentamicin

<sup>1</sup> Conaghan J (2014). pH control in the embryo culture environment. In: Quin P, editors. Culture media, solutions, and systems in Human ART: Cambridge University Press. P. 442-154

CUSTOMER-TECHNICAL SUPPORT  
SUPPORT CLIENTS-SUPPORT TECHNIQUE  
KUNDENDIENST-TECHNISCHER SUPPORT  
ATTENCIÓN AL CLIENTE-ASISTENCIA TÉCNICA  
ASSISTENZA CLIENTI-SUPPORTO TECNICO  
ΑΠΟΙΟ ΑΠΟΣΤΗΡΙΞΗΣ  
ΥΠΟΣΤΗΡΙΞΗ ΠΕΛΑΤΩΝ-ΤΕΧΝΙΚΗ ΥΠΟΣΤΗΡΙΞΗ  
KLANTEN-TECHNISCHE ONDERSTEUNING

FertiPro NV  
Industriepark Noord 32  
8730 Beernem / Belgium  
Tel +32 (0)50 79 18 05  
Fax +32 (0)50 79 17 99  
URL: www.fertipro.com  
E-mail: info@fertipro.com

2797

EN

## FertiCult™ IVF medium

### INDICATIONS FOR USE

FertiCult™ IVF medium is intended for washing and holding of human oocytes, performing fertilization by IVF or ICSI (until 2PN).

For professional use only.

### GENERAL INFORMATION

The medium is complete and needs no further additives. As with all IVF media, FertiCult™ IVF medium has to be pre-incubated in the CO<sub>2</sub> incubator with 5-6% CO<sub>2</sub> for at least 5 hours, but ideally, dishes should be prepared the day before use and incubated overnight (with lid opened) to obtain an optimal pH for oocyte washing/holding.

### COMPOSITION

FertiCult™ IVF medium is a ready-to-use bicarbonate-buffered balanced salt solution, supplemented with glucose (5.55mM), lactate, pyruvate, and human serum albumin (4.0g/l; medicinal substance derived from human blood plasma). The product is also available with phenol red and/or gentamicin (10mg/l; medicinal substance).

### PRODUCT SPECIFICATIONS

- Chemical composition
- pH: 7.20 - 7.50 (37 °C - 6% CO<sub>2</sub>)
- Osmolality: 270 - 290 mOsm/kg
- Sterility test by the current Ph. Eur. 2.6.1 / USP <71>: No growth
- Endotoxin test by Limulus Amebocyte Lysate (LAL) methodology (USP <85>: <0.25EU/ml)
- Mouse Embryo Assay (% blastocysts after 96h after IVF in test medium): ≥ 80%
- Use of Ph Eur or USP grade raw materials if applicable
- Certificate of analysis and MSDS are available upon request or can be downloaded from our website ([www.fertipro.com](http://www.fertipro.com)).

### WARNINGS BEFORE USE

- Do not use the product if:
  - it becomes discolored (if medium contains phenol red), cloudy, or shows any evidence of microbial contamination;
  - seal of the container is opened or defect when the product is delivered;
  - expiry date has been exceeded.
- Do not freeze before use.
- Do not re-sterilize after opening.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/ warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use

### METHOD

#### MEDIUM PRE-EQUILIBRATION/ DISH PREPARATION

The SSCP for FertiCult™ IVF medium describes safety and performance characteristics for the media and is available on the website of FertiPro NV ([www.fertipro.com](http://www.fertipro.com)) or by downloading the following QR code:

Important: Read the note on optimal pH, CO<sub>2</sub> and atmospheric pressure below carefully.

When pre-equilibration is performed in culture dishes, follow the instructions below:

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#### Washing/holding of oocytes and fertilization by IVF or ICSI

After equilibration of the medium, the prepared dishes are ready for use. During oocyte collection, oocytes are identified in the follicular aspirates and then washed before placing into the wells or droplets in the dish. After oocyte collection, oocytes will be checked and may be re-washed, before being placed into fresh droplets/wells for further culture. Generally, within 6 hours of oocyte collection, oocytes will be inseminated by FIV or ICSI with prepared sperm and then left overnight in the incubator.

The presence of 2 pro-nuclei (2PN) is checked 16-20 hrs after fertilization. Once embryos have been identified by the presence of 2PN, they are placed into fresh dishes containing a fresh embryo culture medium (e.g. GAIN™ Medium, FertiPro NV) for further embryo culture and embryo transfer.

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• Conserver le FertiCult™ IVF medium sans gentamicine

Notes:

- 1 Test tube culture is now rare, but for those still using this method, place 1ml of FertiCult™ IVF medium in each tube and then follow the same procedure as for Falcon or Nunc dishes.
- 2 Each laboratory should consult its own validated procedures, optimized for its individual medical program.

#### Note on optimal pH, co<sub>2</sub> and atmospheric pressure

For optimal embryo culture, FertiCult™ IVF medium should be used at a pH of 7.20-7.35. Different factors like the CO<sub>2</sub> concentration in the incubator and atmospheric pressure (which decreases at higher altitudes) have an effect on the pH after equilibration in the incubator<sup>1</sup>.

Therefore, we strongly advise to measure the pH under culture conditions with 6% CO<sub>2</sub> in the incubator in order to determine the CO<sub>2</sub> concentration settings of the incubator which result in an optimal pH of 7.20-7.35.

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