EN

FertiCult™ Flushing medium

INDICATIONS FOR USE

FertiCult™ Flushing medium is intended for *in vitro* procedures involving human gametes (sperm and oocytes), including washing of gametes, sperm swim-up procedures, intra-uterine insemination (IUI) of the spermatozoa and sperm injection during intracytoplasmic sperm injection (ICSI). FertiCult™ Flushing medium can also be used for human embryo washing and holding, and for embryo transfer (ET) in the uterine cavity.

For professional use only.

GENERAL INFORMATION

The medium is complete and needs no further additives. The medium contains HEPES; **no CO₂ incubation is required**. FertiCult™ Flushing medium does not contain heparin, thus for oocyte pick-up, use FertiCult™ Aspiration medium.

COMPOSITION

FertiCult™ Flushing medium is a ready-to-use HEPES-buffered medium which also contains bicarbonate, physiologic salts, glucose, lactate, pyruvate and human serum albumin (4.0g/l, medicinal substance derived from human blood plasma).

The product is also available with phenol red and/or gentamicin (10mg/l, medicinal substance).

PRODUCT SPECIFICATIONS

- Chemical composition
- pH release criteria: 7.30-7.60, pH stability criteria: 7.30-7.90
- Osmolality: 270-290 mOsm/kg
- Endotoxin test by Limulus Amœbozyte Lysate methodology (USP <85>: < 0.25EU/ml)
- Sterility test by the current Ph. Eur. 2.6.1/ USP <71>; No growth
- One-cell mouse embryo assay (% blastocysts after 96 hours, exposure time to test medium: 60 minutes): ≥ 80%
- Human sperm survival assay (% motility compared with control after 24 hours): ≥ 80%
- Use of Ph Eur or USP grade products if applicable
- A certificate of analysis and MSDS are available upon request or can be downloaded from our website (www.fertipro.com)

WARNINGS BEFORE USE

- Do not use the product if:
 - it becomes discoloured (if the medium contains phenol red), cloudy, or shows any evidence of microbial contamination;
 - the seal of the container is opened or defect when the product is delivered;
 - expiry date has been exceeded.
- Do not freeze before use.
- Product which include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.
- Depending on the number of procedures that will be performed on one day, remove the required volume of medium under aseptic conditions in an appropriate sterile recipient. This is in order to avoid multiple openings/warming cycles of the medium. Discard excess (unused) media.
- Keep in its original packaging until the day of use.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE (SSCP)

The SSCP for FertiCult™ Flushing medium describes safety and performance characteristics for the media and is available on the website of FertiPro NV (www.fertipro.com) or by downloading the following QR code:

For further questions regarding to the safety and performance, please contact FertiPro NV for customer or technical support.

METHOD

As this medium does contain HEPES, the medium should not be placed in a CO₂ incubator prior to use. Pre-incubation in incubator (without CO₂) or heating plate is required to ensure that the temperature of the medium is 37 °C at use. Make sure that the lid is closed when bottles are placed in an incubator with CO₂ as this will lower the pH to 6.90-6.95.

Each laboratory should consult its own validated procedures, optimized for its individual medical program. When performing IUI or ET, follow the instructions of the specific catheters used. In addition, some additional information on the following procedures is provided below:

Washing of spermatozoa (suggested procedure)

The washing of spermatozoa can be done at room temperature or at 37 °C.

- Add 5ml FertiCult™ Flushing medium to the native semen sample and mix. Centrifuge for 5-10 minutes and approximately 300g.
- Remove supernatant and leave about 0.5ml of semen in the centrifuge tube.
- Add 5ml FertiCult™ Flushing medium to the test tube. Mix the solution gently until the pellet is completely dissolved.
- Centrifuge again for 5-10 minutes at 300g.
- Resuspend in a suitable volume of FertiCult™ Flushing medium.

Swim-up procedure (according to WHO, 2021)

- Gently layer 1.5ml FertiCult™ Flushing medium over

1ml of semen in a conical based centrifuge tube. Incline the tube at an angle of 45° and incubate for 1 hour at 37 °C.

- Gently turn to the upright position and remove the uppermost 1ml.

- Dilute this aliquot of motile cells with 2.5ml FertiCult™ Flushing medium. Centrifuge for 5-10 minutes at 300g, remove supernatant and finally resuspend in 0.5ml of FertiCult™ Flushing medium.

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